

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

*This product meets or exceeds all federal safety and environmental regulations.

R250130

Bari+Max
Support Where You Need It

Item **10169**

Bariatric HD Steel Walker

User's Manual



Not made with natural rubber latex.

Manufactured for: Dynarex Corporation
11 Dynarex Drive | Middletown, NY 10941
USA | www.dynarex.com | Made in China
Symbol Glossary: dynarex.com/symbols.php

dynarex 
we care like family™

This Walker is equipped with four standard legs and two legs with wheels to allow the walker to be used as a standard walker or rolling walker.


HEIGHT ADJUSTMENTS

- Stand with your arms hanging loosely at your sides.
- The walker hand grips should be at the height of your wrists.
- The walker should be adjusted with push buttons and holes on all four legs.
- Push the button with your thumb and adjust to the correct hole height.
- Repeat this adjustment for all four walker legs making sure that the walker is level.
- Make sure that each push button is locked in place after the height is adjusted.

CAUTION

Be aware of the following:

- Electrical cords in the way
- Liquids on floor
- Throw rugs
- Toys or debris
- Furniture
- Doors and doorsteps
- Gaps in floor

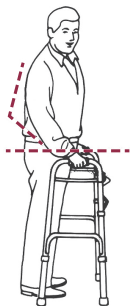

WARNINGS

1. Wear shoes or slippers with rubber or other non-skid soles.
2. Use Dynarex accessories only. Use of other manufacturers' accessories can result in improper fit, and risk fall and/or injury.
3. Do not use on stairs or escalators.
4. Check the tips and wheels of your walker daily and replace them if they are worn.
5. Keep space between you and your walker, while keeping your toes inside your walker. Stepping too close to the front or tips or wheels may make you lose your balance.

WARNINGS (CONT'D)

6. Do not exceed the maximum weight capacity of 700 lb.
7. Do not use this walker in pools, showers, or other water environments, as this could result in corrosion of parts, damage to the walker, and possible injury to the user.

MAINTENANCE

Clean grips with mild soap or detergent. Never use organic solvents that could dissolve the grips.

WALKING WITH THE HEAVY-DUTY STEEL WALKER

THREE-POINT GAIT:



Step 1

Align the middle of your foot with the back legs of the walker.



Step 2

Standard walker: Lift the walker up and place it comfortably in front of you, making sure all four legs end up on the floor.

Wheeled walker: Push the walker forward slowly and smoothly.



Step 3

Carefully, walk forward one step at a time.

Weight capacity 700 lb.