

Reorder No. 10524

Bari+Max^{dynarex®}

Support Where You Need It



Bariatric Geri Chair Recliner 3-Position • Blueridge



 Manufactured for: Dynarex Corporation
10 Glenshaw Street • Orangeburg, NY 10962
USA • www.dynarex.com

Made in China

CLINICAL CARE RECLINER

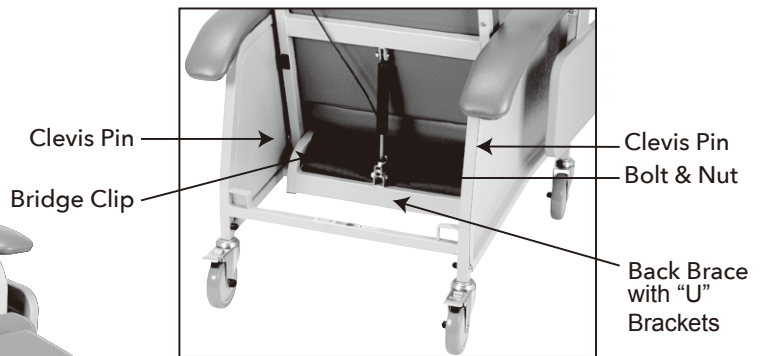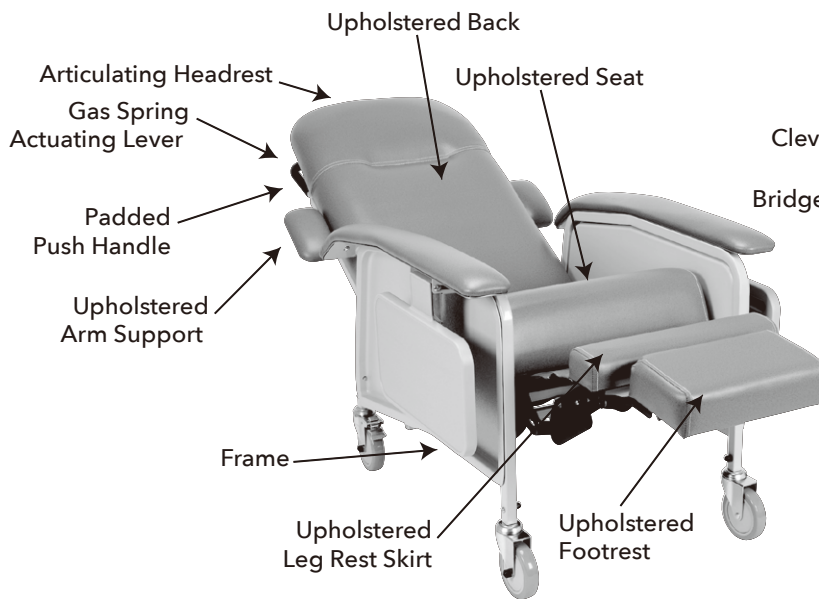

IMPORTANT SAFETY GUIDELINES:

- Weight Capacity: 300 lb.
- Casters must be locked except during transport.

WARNING/CAUTION

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a health-care professional, dealer or technical personnel before attempting to use this equipment otherwise, injury or damage may occur.
- ALWAYS keep hands and fingers clear of moving parts to avoid injury.
- DO NOT use near an open flame or heat source, as all upholstery materials, even though treated with flame retardant, can ignite when exposed to open flame.
- Rear caster wheels MUST be locked during user transfer to or from recliner and while tilting and/or reclining or inclining (sitting up).
- Ensure that casters are free of debris.
- DO NOT enter or exit the recliner when footrest is elevated.
- DO NOT sit on the footrest when recliner is extended. The recliner will tip and bodily injury may occur.
- DO NOT push or pull footrest to position the recliner.
- DO NOT place hands or feet into any openings when adjusting the recliner. Attendant or care provider should always verify placement of user's hands and feet prior to adjusting the recliner. Failure to do so may result in serious bodily injury.
- Operation of recliner with missing or broken footrest or recline mechanism guards may result in bodily injury. Periodic inspection of the footrest guard and recline mechanism guard MUST be made four times a year. Torn, cracked or otherwise damaged guards MUST be replaced prior to use. Recliner MUST be removed from service until replacement guards are installed.
- DO NOT lift the recliner by the footrest. Lifting by means of the footrest may result in injury to the user and/or damage to the recliner.
- DO NOT attempt to stop the recliner with the wheel locks, while in motion. Wheel locks are not brakes.
- Before attempting to transfer in or out of the recliner, every precaution should be taken to reduce the gap distance. Turn the recliner toward the object you are transferring onto. When transferring to and from the recliner, ALWAYS engage both wheel locks. Special care MUST BE taken with people that have physical limitations which may require an assistant.
- DO NOT traverse, climb or go down ramps or slopes greater than 9°.
- Test the recline feature of the recliner first without an occupant to ensure the gas cylinder is operational. DO NOT operate the recline option if the gas cylinder is not operational.
- ALWAYS make sure that the recliner is stable before using the recline option. Make sure the patient is properly positioned in the recliner before reclining or inclining to maintain maximum stability and safety.
- Excessive force or effort is never necessary to change the seating position of the chair. Do not operate recliner by applying force or pressure to leg rest.
- When returning the occupant of the recliner to the full upright position, more body strength will be required for approximately the last twenty degrees of incline (sitting up). Make sure to use proper body mechanics (use your legs) or seek assistance to avoid injury.
- Physical limitations of a user and larger occupants of the recliner can present conditions that can injure the assistant. When in doubt, assure that additional assistants are present before operating the recline feature.
- Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Dealer for further instructions.

ASSEMBLY INSTRUCTIONS

NOTE: After removing all components from the carton, engage the two caster locks before beginning assembly..

Upholstered Back:

1. Remove cardboard shipping retainers from around Gas Spring Cylinder. Rotate the hinge on the gas spring so that it is perpendicular to the upholstered back. (See figure 1)
2. Lift the upholstered back from the sides and place it horizontally with the upright back tubes resting on the recliner frame. (See figure 2)
3. Undo hook and rings on side panels, both sides so mounting holes are exposed. Align the holes on the upright back tubes of the upholstered back with the holes on the seat frame. Replace the clevis pins through the mounting holes.
4. Secure the two clevis pins by replacing the bridge clips.

5. Raise the upholstered back to vertical position. Tuck the plastic skirt at rear of upholstered back behind cross brace of seat frame. Align the hinge at the bottom of the gas spring with the holes in the "U" bracket in the seat frame cross brace. While holding the back with one hand, secure the gas spring hinge to the "U" bracket by replacing the bolt with the other hand. Secure the bolt by replacing the nut. (See figure 3)

Seat Frame:

1. Align holes in arm support mount to mounting holes in seat frame. (See figure 4)
2. Insert screws through arm support mount and seat frame holes. Secure screw with nut.
3. Repeat for other side.

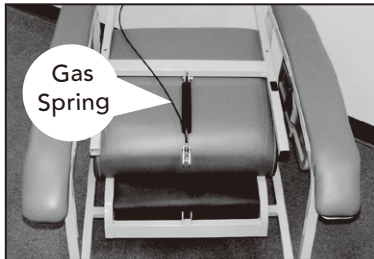


FIGURE 1

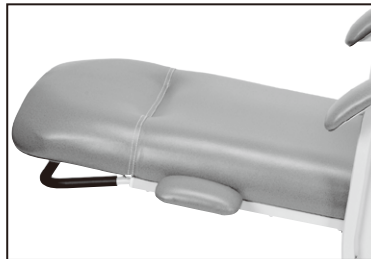


FIGURE 2

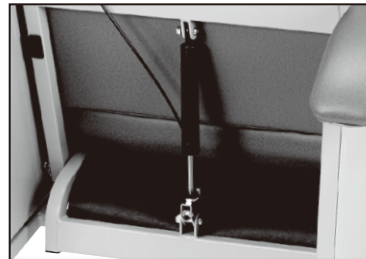


FIGURE 3



FIGURE 4

SIDE TABLES AND HEADREST

Side Tables:

1. To place either table in raised position: raise table all the way up and slide in until tabs engage slots in mount bracket. Press down lightly on table to ensure lock tabs are engaged.
2. To return table to self-storing position: Lift table up until tabs disengage from slots. Slide out and allow table to lower. (See figure 5)

Articulating Headrest:

Articulating headrest may be adjusted to a comfortable position by pulling forward or pushing back.

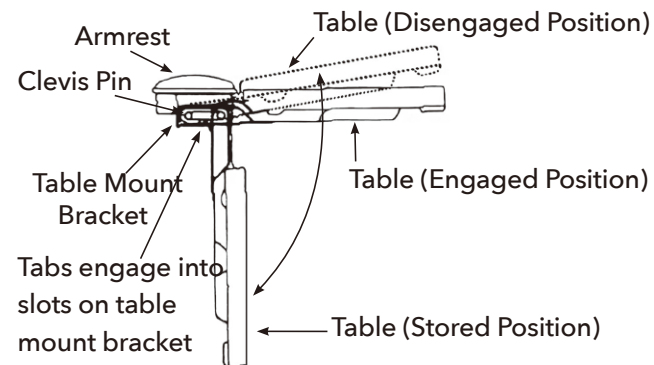


FIGURE 5

SEAT POSITIONING

Adjusting the recline:

Lock the rear casters. While holding the push bar, squeeze the piston spring release handle. To lower, push back on padded push handle. To raise, pull forward on padded push handle. To achieve the Shock Position, follow the above steps and continue to lower the back until the shock position pictured is achieved.



Recline Position 1




Recline Position 2



Shock Position


CASTER OPERATIONS

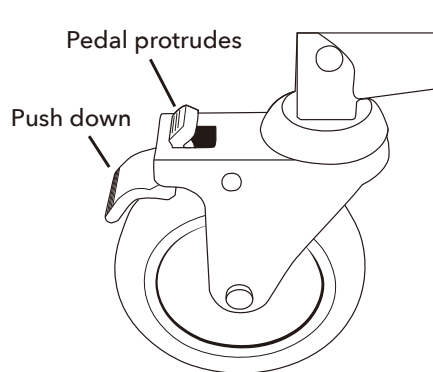

TOTAL LOCK: Step on the rear pedal of the caster until the pedal engages. The front of pedal will protrude from wheel cover.

UNLOCK: Step on the protruding front part of the pedal to disengage lock to unlatch.

NOTE: This chair is equipped with four swivel casters for improved maneuverability. The rear casters are also equipped with locking features.

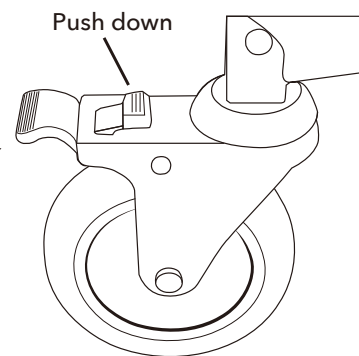

MAINTENANCE/CLEANING

Every Three Months:

- Check and clean casters.
- Tighten nuts.
- Check brakes on casters.
- Check and clean lock recline mechanism.

For upholstery maintenance and stain removal, the following hints will be helpful:

- Ordinary Dirt: Wash with warm water and a mild soap or detergent.
- Stubborn Dirt: Using a soft bristle brush, follow directions above.

- Ground-in Dirt: Scrub with powdered cleanser or similar detergent.
- Chewing Gum: Scrape carefully.
- Nail Polish and Remover: Blot immediately to prevent damage.
- Ball Point Pen: Blot immediately with a white cloth dampened in water or mineral oil.
- Waxing and Refinishing: Improve wearability. Use any quality paste wax.

NOTE: A diluted solution of 10% (or less) of bleach or Clorox® may be used when cleaning this chair.

WARRANTY

Your Dynarex Product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced

- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.



SYMBOL GLOSSARY
For an explanation of symbols
used in Dynarex packaging, visit
dynarex.com/symbols.php