

LIMITED LIFETIME WARRANTY

Your Dynarex Product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions:

- Problems arising from normal wear
- Problems arising from failure to adhere to the product instructions
- Problems arising from misuse, negligence, accident or improper operation, maintenance or storage
- Problems arising from modifications or unauthorized repairs, parts or attachments
- Products where the serial number has been removed or defaced
- Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement

Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

If you have a question about your Dynarex device or this warranty, please contact an authorized Dynarex dealer.

*This product meets or exceeds all federal safety and environmental regulations.



WARNING:
Cancer and Reproductive Harm
www.P65Warnings.ca.gov

R260416

Weight Capacity: 500 lb.

Not made with natural rubber latex.



Manufactured for: Dynarex Corporation
11 Dynarex Drive | Middletown, NY 10941
USA • www.dynarex.com | Made in China
Symbol Glossary: dynarex.com/symbols.php



Item 10203

Bari+Max
Support Where You Need It

Titan-500 Bariatric Rollator




Important Safety Notice

Do not have anyone push you while you are seated on the rollator. This is a walking aid only and is not to be used as a transportation device.

ASSEMBLY INSTRUCTIONS

Do not adjust or operate rollator without carefully reading all instructions in this manual.

1. First remove all contents from carton: backrest, handle with tube, rollator frame and basket.
2. Open the frame. Ensure that the folding hinge is locked into place. Slide the backrest into frame brackets making sure snap bottoms protrude through holes.
3. Unscrew height adjustment knob from the frame and insert handle tube into frame at desired height, lining up holes appropriately. Slide bolts through holes from the inside and screw on tightening knob until secure.

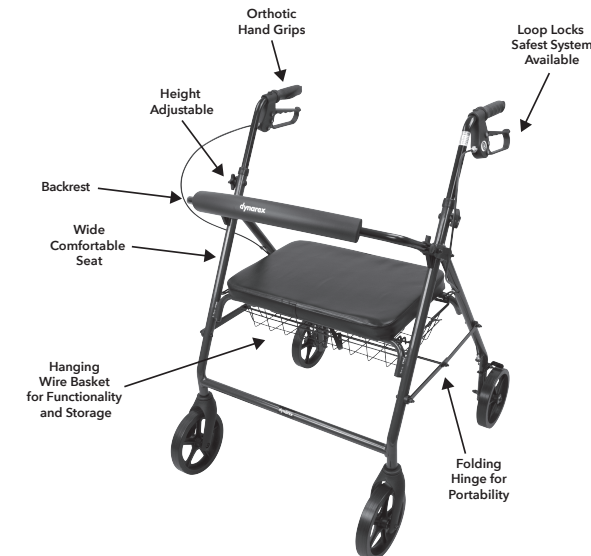

SAFETY PRECAUTIONS

1. Please make sure the rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the rollator. When wheels are locked (see illustrations for locking and releasing locks), lock bar on rear wheels should prevent wheels from turning. If the wheels turn while the brake is engaged, follow

the instructions to adjust the hand brake system. When locks are released, the wheels should turn freely.

4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.

ASSEMBLY INSTRUCTIONS

Always engage and disengage wheel locks on both sides simultaneously.

1. To slow down or come to a temporary stop – pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To temporarily lock wheels in position – press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked. Always lock the wheels when using the rollator as a seat.
3. To release wheel locks – push up on both loop locks with palms of hands to original position.



INSTRUCTIONS TO ADJUST THE HAND BRAKE SYSTEM

Important Safety Notice

The hand brake system fitted on this rollator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance setting. To better suit you personally, please follow the instructions set out on left side. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

