

ELONGATED RAISED TOILET SEAT



USER WEIGHT LIMIT: 300 lb.



Manufactured for: Dynarex Corporation
11 Dynarex Drive • Middletown, NY 10941
USA • www.dynarex.com

Symbol Glossary: dynarex.com/symbols.php

Made in China

R241212

 **WARNING:**
Cancer and Reproductive Harm
www.P65Warnings.ca.gov



Not made with natural rubber latex

INCLUDED IN BOX



Elongated Raised
Toilet Seat



Screw
x2



Wing Nut
x2




Washer (A)
x2




Washer (B)
x2

INSTALLATION

1. Remove your toilet seat.

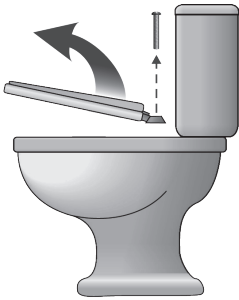

2. Place your Elongated Raised Toilet Seat on the toilet, place your toilet seat on top of the riser, lining up the holes of the riser and the seat with the holes on the toilet frame.

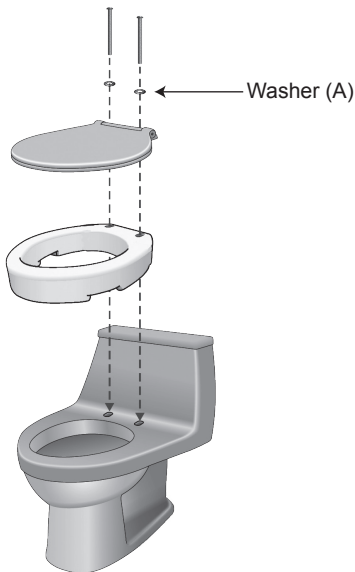

3. Put the enclosed bolts through the holes, attaching the Elongated Raised Toilet Seat to the toilet frame with the nuts provided.



WARNING! Before using your Elongated Raised Toilet Seat, push on the unit firmly to ensure that it is securely attached.